

# 1

### Open

- Open the ACCUSOL overpouch from any corner
- Place the ACCUSOL bag flat with the connector towards you
- Check solution formulation and expiry date
- The ACCUSOL bag may feel damp as a result of the sterilisation process



# 5

### Break the short seal

- Break the short seal by lifting the large chamber, grabbing the bag firmly in both hands and rolling it toward the connector
- Apply pressure, by leaning over the bag, and pushing the solution towards the short seal to break it
- Check that the short seal has been fully opened
- Recheck that the bag has no leaks



# 2

### Check!

- ACCUSOL is clear and free from particles
- Bag has no leaks
- Wings of the bag connector are in open position and cap in place
- Short and long-seals are intact (press down on the upper, then lower chamber); ACCUSOL should never be used if either seal is already broken
- Always use aseptic technique during the whole procedure



# 6

### Hang the bag

- Remove the cap of the bag connector
- Connect solution line
- Use only with compatible Luer connectors
- Compatible manifold set can be used to connect multiple Accusol bags to the solution line



# 3

### Inject

- Remove blue pull-ring and disinfect medication port
- Inject prescribed additives
- Additives must be injected before opening the long-seal



# 7

### Start the flow

- Hold the bag connector halfway down the wings
- Squeeze until the wings securely clip onto the main body of the connector



# 4

### Break the long-seal

- Position ACCUSOL with the bag connector facing away from you
- Grasp the two sides of the large chamber
- Squeeze the sides inwards while rolling your wrists under the bag to open the long-seal
- Press down on the upper & lower chambers to complete opening and mixing
- Check that the long-seal has been fully opened
- Recheck that the bag has no leaks and is free from particles



### IMPORTANT INFORMATION

- The medication port must be disinfected prior to injecting additives
- Additives must be injected before opening the long-seal
- Compatibility must be checked before admixture. Use the solution immediately after the addition of additives
- The short and long-seals must be opened before connecting the solution line
- ACCUSOL must be used within 24 hours of mixing
- ACCUSOL bag is for single-use only. Discard any unused fluids. Discard the device according to your local regulations after use, do not use it as an effluent collection bag
- Do not use the product after date of expiry
- Do not carry ACCUSOL by overpouch
- Do not lift the bag using the bag connector
- To prevent accidents, store carton boxes at a low level

*For the duration of therapy, the dialysate line should be regularly inspected to verify that the solution in the tubing is clear. If particles are observed, the ACCUSOL solution and filtration lines must be replaced immediately and the patient monitored.*